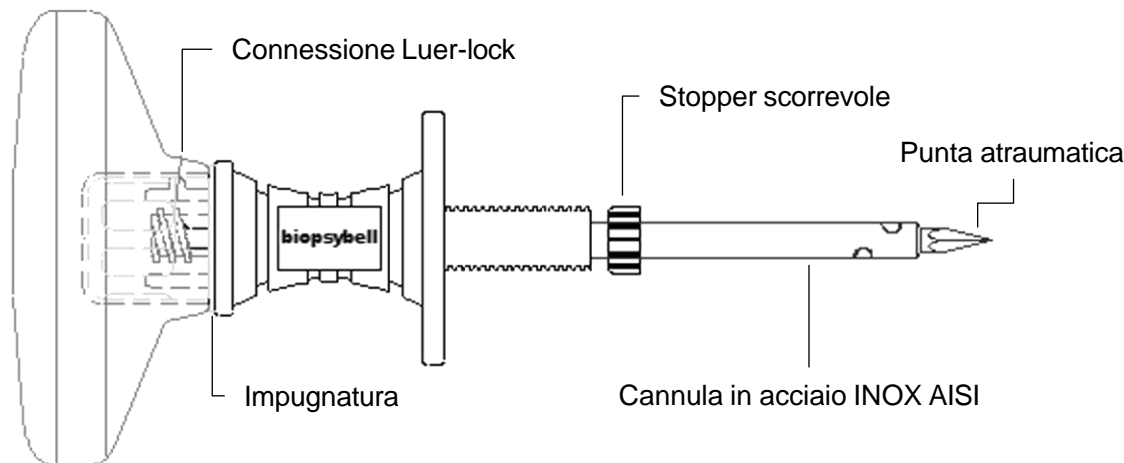
## INTRAOX<sup>TM</sup>

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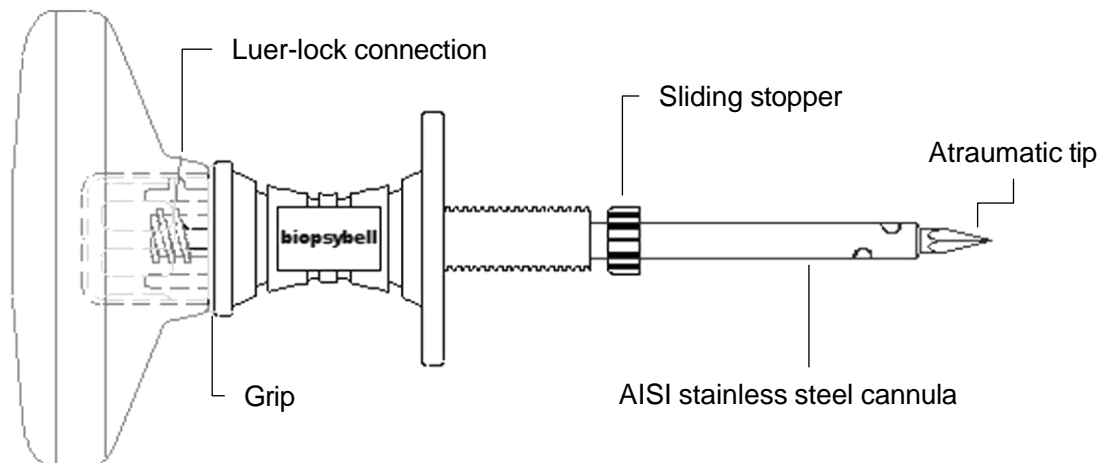
**EN INTRAOX™ - Intraosseous infusion needle**

**INSTRUCTIONS FOR USE:**

- Before using INTRAOX™, make sure that the packaging is not damaged.
- Check that the cannula is not damaged and, if necessary, contact the holder of the notification in Brazil.
- Remove INTRAOX™ from the packaging, taking care to operate under aseptic conditions.
- Remove the protection.
- Perform the skin disinfection procedure and, if necessary, apply local anesthesia.
- Hold the handle and position the tip of the cannula in the application area. The appropriate area for intraosseous infusion is the anteromedial surface of the proximal tibia, below the tibial tuberosity (children under 6 years of age). Another appropriate area for infusion is the distal tibia 1-2 cm above the medial malleolus (children over 6 years of age or adults), the distal femur and the distal humerus.
- After penetrating the skin and subskin, push forward by rotating to the right and left until penetrating the bone cavity to the preset depth.
- Remove the stylet.
- Insert a standard syringe into the Luer-Lock connection and proceed to infuse the necessary fluids.

**AREAS FOR APPLICATION:**

The recommended place for INTRAOX™ infusion is:

- Children under 6 years of age: The anteromedial and proximal tibial surface, immediately below the tibial tuberosity.
- Children over 6 years of age and Adults: In addition to the locations indicated above, the distal tibia 1-2 cm above the medial malleolus and the distal femur and distal humerus.

**INDICATIONS:**

INTRAOX™ should be used for the administration and infusion of fluids at the intraosseous level in emergency situations where traditional vascular access is impossible or when it is not possible to obtain it in a short time. It must be used exclusively for medical control. Use intraosseous infusion only as a transitional access for the infusion of liquids.

**RECOMMENDATION:**

INTRAOX™ should remain in the patient for a transient period of time as recommended by the manufacturer after its intraosseous infusion.

**OPERATING PRINCIPLE:**

The cannula has a piercing tip and a Luer-lock connection on the handle to attach a standard syringe for infusing substances. (In accordance with ISO 80369).

## **EN** INTRAOX<sup>TM</sup> - Intraosseous infusion needle

### **PRECAUTIONS FOR USE:**

The INTRAOX<sup>TM</sup> cannula must be used exclusively by physicians.

Reprocessing after use is prohibited and disposal is mandatory, as the cannula may pose a risk of contamination and/or infection.

It is also recommended not to use the device if particularly risky manoeuvres are required in which the needle must be subjected to twisting forces or bends with a narrow radius that could cause the cannula to rupture. Therefore, handle the product with care and dispose of it in compliance with current regulations and medical practice.

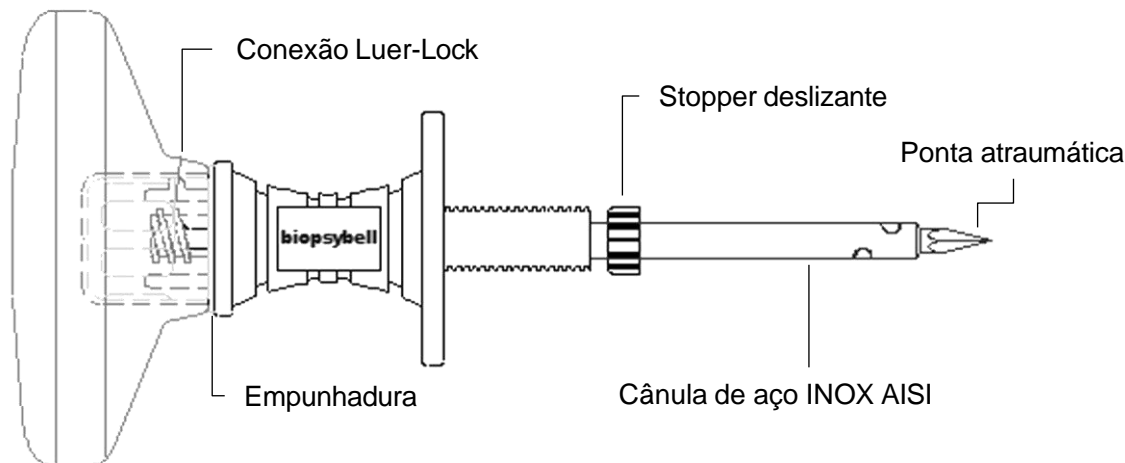
### **PACKAGING :**

The needle is individually packed in a medical-grade paper and transparent laminated film bag and is sterilized by ethylene oxide (EtO) in compliance with ISO 11135 requirements. It is sold in packs of 10 pieces. The device is not reusable. If it is reused, there is a risk of altering device performance and/or the possibility of cross-contamination. Do not use the device if the package is open, damaged, or contaminated.

### **STORAGE, TRANSPORT AND HANDLING CONDITIONS:**

The product is not subject to specific humidity or temperature conditions. Keep away from heat sources. Handle with care to avoid compromising the primary or secondary packaging of the device. Storage, transport and handling should be carried out in accordance with the symbols on the label and maintain the integrity of the product and its packaging.

*Biopsybell declines all responsibility for any damage caused by improper use or different from what is indicated in this instruction leaflet.*

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